

Implementation of personal Logo for Printout Pages

[Go to start of metadata](#)

Introduction

This feature allows you to print your own logo on the printouts. It was first implemented in custo diagnostic 4.3.x

Requirements

- The logo must be in BMP format
- Size: The height must not exceed 100 pixels, and the width must not exceed 300 pixels
- Starting from custo diagnostic 5.x you can use JPG and PNG
- Starting from custo diagnostic 5.10.11 the Printout will be printed with custo med logo as default

Settings

Setting	Default	WS	US	G	S	Description
Drucken /Darstellung\PrintBitmapHeight	0	✓	✓			Height of the bitmap in hundredths of a unit, maximum 300 (0 = the height is calculated automatically: 100 pixels correspond to 3 units). The width is adjusted accordingly.
Drucken /Darstellung\PrintBitmapPathH ochFormat	<i>leer</i>	✓	✓			Bitmap including path, to be printed on the portrait-oriented print head
Drucken /Darstellung\PrintBitmapPathQ uerFormat	<i>leer</i>	✓	✓			Bitmap including path, to be printed on the landscape-oriented print head
Drucken /Darstellung\PrintBitmapRight Aligned	false					false = The bitmap is printed left-aligned at the print head true = The bitmap is printed right-aligned at the print head



Please Note

The settings are specific to each user and workstation! (WS = Workstation specific, US = User specific)

If you want to disable the printout of the Logo you can set "PrintBitmapHeight" to "-1".